

Rejuvapod™

WORKOUT-FREE FITNESS & WELLNESS

ADDENDUM ACA SURROUND SYSTEM



Owner's Manual

PN 206823-ACA RJ.10.0421.1.U

WWW.WELLNESS-USA.COM

AMBIENT COLOR ARRAY SURROUND SYSTEM

The Rejuvapod includes an integrated Ambient Color Array Surround (“ACA Surround”) System. The ACA Surround System is multicolor and utilizes yellow, red, blended gold-orange and ambient color array near-infrared non-thermal lamps.

The ACA Surround System can be programmed to provide bright “Flashing” or “Pulsing” Lights, similar to flashing in video games, computer games, light stimulus and/or other flashing devices. Certain persons can be adversely affected when exposed to flashing rates that exceed 20-32Hz rate, which could result in headaches, nausea or, in rare instances, seizures; such persons should not use the ACA Surround System.

Photosensitive Flashing Lights Seizure Warning

A small percentage of people may experience a seizure when exposed to flashing or pulsing lights. Even people who have no history of seizures or epilepsy may have an undiagnosed condition that can cause these “photosensitive episodes.”

These adverse episodes may have a variety of symptoms, including lightheadedness, altered vision, eye or face twitching, jerking or shaking of the arms or legs, disorientation, confusion or momentary loss of awareness. Seizures may also result in loss of consciousness or convulsions that can lead to injury from falling down or striking nearby objects.

Immediately stop using the unit and consult a doctor if you experience any of these symptoms. If you or any of your relatives have a history of seizures or epilepsy, consult a doctor before using the Rejuvapod.

1.1 Operational Safety

CAUTION:

- Do not touch the lights.
- If you have any of the contraindications listed in the safety sections, the contraindication section or section 4 of the Rejuvapod Owner’s Manual or other health concerns such as in poor health, asthma, problems with breathing, low or high blood pressure, please consult your physician prior to using this machine.
- Children under the age of 16 must have adult guidance.
- Recommended session duration is 60 minutes or less.
- To avoid high temperature contact, use wooden handle above hand ports to open and close device.
- Always wear protective sys glasses or shields.

- Please remove any metal items, such as rings and jewelry, before beginning a session.
- Please read Owner's Manual and Addendum before using.
- Never remove or dismantle array or hood unless it is being disassembled by a professional technician.
- If you have any questions, concerns and/or problems, please contact your nearest dealer immediately.

For operational safety, strictly observe the following instructions:

- Do not sit or place heavy objects on top of the hood.
- Do not sit on the contour bed when hood is opened halfway.
- Do not sit or raise up without avoiding ACA Surround System.
- Do not push up or pull down with force.

1.2 Preparation for Operation

- Check to see if the power cord plug is plugged into the appropriate electrical wall outlet. Please refer to **Installing Bed** in the owner's manual **SETUP** for further details.
- Open the hood fully, then partially raise the contour bed to locate the ON/OFF switch on the back of the system.
- Sit on the contour bed. Then move slowly to recline.
- Grasp handle inside the hood and gradually pull it downward.
- Adjust the Velcro closure of the neck drape to cover opening completely.
- Push hands through the hand ports to operate the Control Panel Keys.

1.3 Operation of the Ambient Color Array Surround System


Note: Before turning the system on, be sure the ACA Surround System is connected to the capsule.

Touch Screen Operation


The system's ambient color array operating parameters can easily be set and modified by using the convenient touch screen. Adjusting the operating parameters must be done from outside the capsule; adjustments cannot be made while inside the capsule.

1. Turn On/OFF ACA Surround Panel

- a. Make sure the Rejuvapod is turned on.
 - b. Flip on/off the secondary switch located behind the ACA Surround System box.
 - i. In case of emergency the unit can be switched OFF by the user (use the switch located behind the light box).
- 2. Positioning the ACA Surround System**
- a. Once the user has been comfortably positioned in the capsule, gently lower the ACA Surround panel and center the unit using the user's nose as a reference. The panel should be placed from 8-10 cm surrounding and above the user's face.


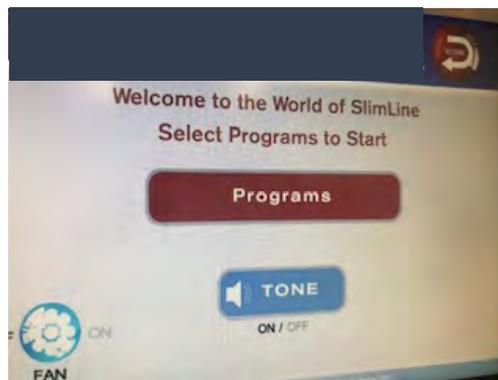


- 3. Selecting your ACA Surround Program**
- a. Select your choice of relaxing ambient mood program from the screen shown below. You can move back and forth between the screens by using the  at the upper right hand corner of the screen.

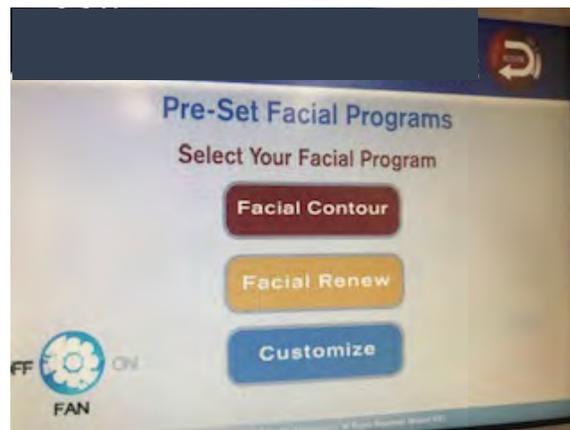

- 4. Adjusting the Parameters Utilized in your Program**
- a. Except for the ambient color selected, all the parameters can be adjusted

by using the  and  to the left and right of each feature shown below.

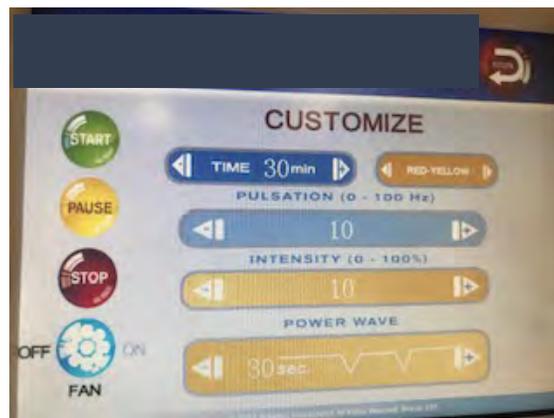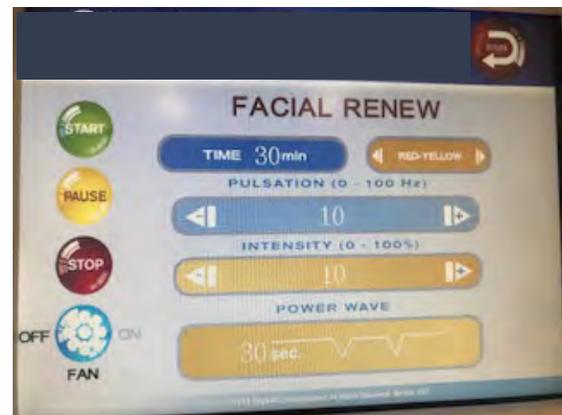

5. Modifying the Status of an ACA Surround Session

a. To modify the status of a session while in use, press the following keys on the control panel of the ACA Surround:

- i. **START**: To commence a session
- ii. **STOP**: To halt a session
- iii. **PAUSE**: To interrupt a session; to continue press **Pause** again

1. Ambient Array Multi-Color(s)

a. Options: Yellow, red or blended gold-orange.

2. Timer

a. Choices range from 0 to 30 minutes.

3. ACA Surround Repetition

a. 1 to 100

4. **Ambient Color Brightness**
 - a. Variable (adjust from 1 to 100%)
5. **ACA Surround Wave**
 - a. 0 to 30
6. **Cooling Face**
 - a. Fan: On/Off

1.4 Supplied Accessories

The ACA Surround System is supplied with the following components:

- ACA Surround System
- Attachment Arm Kit
- Eyewear (for user) – 1 pair

NOTE: Supplied accessories are subject to change. Contact your supplier with questions.

1.5 Operating Conditions for ACA Surround System

- Normal working conditions:
 - Ambient temperature: 5 to 30° Celsius (48 to 86° Fahrenheit)
 - Relative humidity: ≤70%
 - Atmospheric pressure: 860-1060hPa
 - Required power source: Single-phase 110/220V, 10/16A
- Power Supply Distribution
 - Electrical power: AC 100-230V, 50/60Hz, DC 12V


1.6 Transportation Conditions for ACA Surround System

- Normal transportation conditions:
 - Ambient temperature: -28.89°- 37.2 Celsius (-2 to 99° Fahrenheit)
 - Relative humidity: 50-80%
 - Atmospheric pressure: 860-1060hPa

1.7 ACA Surround System Attachment Arm Adjustment

- a) Adjust the Ball-Joint for Tilt Action. Adjusting may be required to the ball joint mechanism. If the Light Box doesn't hold its position or is too tight, adjust the four tensioning screws located around the ball joint cover (clockwise to tighten, counter-clockwise to loosen). Check the Light Box and adjust again if necessary.

Screws

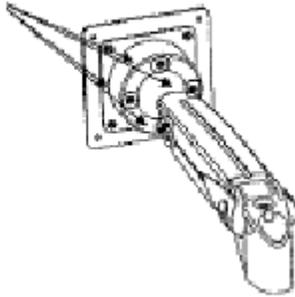


- b) Vertical Movement. Adjusting might be required. Use the wrench provided in the accessory kit. If the arm tends to drift down or rise up when positioned, it will be necessary to make a small adjustment in the adjustment screw (see graphic bellow).

Adjustment Screw



Rising: Adjust the screw moving the wrench clockwise till you reach the desired position. If the unit moves down, adjust the crew moving the wrench counter-clockwise till you reach the desired position.

Note: When performing this adjustment, the arm must be in a horizontal position.

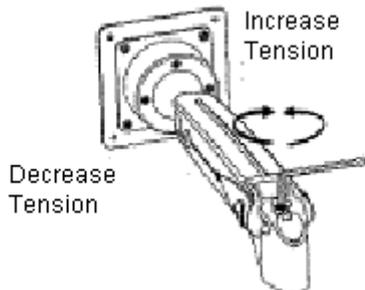

Note: To prevent damage to the arm, remove the wrench before releasing the arm from the horizontal adjustment position.